



safe&care

IN ANY ACTIVITY OR PROFESSION
SAFETY MUST BE FACED IN A WAY THAT
SERIOUS AND RESPONSIBLE

PREVENTING THE PROPAGATION OF COVID-19

safe&care

SURGICAL GOWN

DISPOSABLE

The manufacturer must accompany the gown with a brochure containing information for the end user, in particular information to allow appropriate use of the gown.

Name of manufacturer: VIEIRA POCARGIL SA
Address: Zona Industrial de S.Martinho do Campo, 4830-102 Povoá Lanhoso
Commercial brand or code/Ref.ª: BCDP1 #SAFE&CARE
Lot n.º. | Month | Year of manufacture: LOT1 /MAIO 2020
Composition of raw-materials: 50% POLYESTER 50% POLIPROPILENO
Size identification (child/adult): Adult M



CONFECTION COMPLIANCE NORM

TEST/NORM: CONFECTION CONFORMITY / MI

RESULTS:

REF mask. BCDP1 #SAFE&CARE

The surgical gown shows conformity of confection.





BURST RESISTANCE

TEST/NORM: BURST RESISTANCE/ EN ISO 13938-1:1999 - HUMID

TEST/NORM: BURST RESISTANCE/ EN ISO 13938-1:1999 - DRY

BREAKING STRENGTH AND ELONGATION

TEST/NORM: BREAKING STRENGTH AND ELONGATION (STRIP METHOD)

EN 29073-3:1992 - HUMID

TEST/NORM: BREAKING STRENGTH AND ELONGATION (STRIP METHOD)

EN 29073-3:1992 - DRY

WATER PENETRATION RESISTANCE

TEST/NORM: RESISTANCE TO WATER PENETRATION (HYDROSTATIC PRESSURE) IN

ISO 811:2018

BACTERIAL PENETRATION RESISTANCE

TEST/NORM: RESISTANCE TO BACTERIAL PENETRATION - MEDICAL DEVICES / ISO

22610:2006



INSTRUÇÕES DE USO

The surgical gown must be handled correctly.

The following are recommended instructions for the correct use of the surgical gown:

- a)** Hygiene your hands with soap and water or an alcohol-based solution before wearing the surgical gown.
- b)** Put on the surgical gown with the internal side facing the body, and the external side facing out.
- c)** Adjust the surgical gown to the body, taking into account the cover of the upper and lower limbs and the proper adjustment at the ends, closing all the adjustment systems.
- d)** The removal of the surgical gown should be done so as not to touch the body by turning the suit inside out.
- e)** The surgical gown should be placed in a suitable hospital waste container.
- f)** The hands shall be cleaned again at the end of the use and handling of the surgical gown.

CARE AND MAINTENANCE INSTRUCTIONS

Products awaiting use must be stored in a dry, enclosed place and protected from potential contamination.

In case of defect or damage do not use the product, replace with another.

PRODUCT LIFETIME

The product is for single use only and should therefore be marked with the warning phrase: "DO NOT REUSE".



safe&care

WWW.SAFEANDCARE.PT

geral@safeandcare.pt

Rua do Parque Industrial n°366

4830-103 Campos PVL

PORTUGAL

T: (351) 253 639 280

FAX: (351) 253 632 600