



**safe&care**



## **AN ENVIRONMENTAL CONCERN FOR A GREENER FUTURE**

**In any activity safety must be taken seriously and responsibly.**

The negative impacts that human beings provoke on nature are becoming more and more evident. Now more than ever, in this period of the new coronavirus pandemic, it is important to create sustainable practices.

**safe&care**

# PROTECTIVE COVERALL CONTROLLED ENVIRONMENT REUSABLE **25 WASHES**

The manufacturer must accompany the coverall with a brochure containing information for the end user, in particular information to allow appropriate use of the coverall.

Name of manufacturer: VIEIRA POCARGIL SA  
Address: Zona Industrial de S.Martinho do Campo, 4830-102 Povoá Lanhoso  
Commercial brand or code/Ref.ª: FRP1 #SAFE&CARE  
Lot n.º. | Month | Year of manufacture: LOT1 /MAIO 2020  
Composition of raw-materials: 50% POLYESTER 50% POLIPROPILENO  
Size identification (child/adult): Adulto (S, M, L, XL)



## CONFECTION COMPLIANCE NORM

TEST/NORM: CONFECTION CONFORMITY / MI

### RESULTS:

REF mask. FRP1 #SAFE&CARE

The protective coverall shows conformity of confection.





### BURST RESISTANCE

TEST/NORM: BURST RESISTANCE/ EN ISO 13938-1:1999

### BREAKING STRENGTH AND ELONGATION

TEST/NORM: BREAKING STRENGTH AND ELONGATION (STRIP METHOD) EN 29073-3:1992

### WATER PENETRATION RESISTANCE

TEST/NORM: RESISTANCE TO WATER PENETRATION (HYDROSTATIC PRESSURE) IN ISO 811:2018



### TREATMENT AND WASHING

TEST/NORM: PRELIMINARY TREATMENT - INDUSTRIAL WASHING AND FINISHING EN ISO 15797:2018

The protective coverall should be washed under the following conditions:

- Washing cycle 25 (temperature 75°C), in an industrial type machine, according to NP EN ISO 15797:2017
- Drum drying.



### USER INSTRUCTIONS

The protective coverall must be handled correctly.

The following are recommended instructions for the correct use of the protective coverall:

- a) The protective coverall should be washed before each use.
- b) Hygiene your hands with soap and water or an alcohol-based solution before wearing the protective coverall.
- c) Put on the protective coverall with the internal side facing the body, and the external side facing out.
- d) Adjust the protective coverall to the body, taking into account the cover of the upper and lower limbs and the proper adjustment at the ends, closing all the adjustment systems.
- e) The removal of the protective coverall should be done so as not to touch the body by turning the suit inside out.
- f) The protective coverall should be placed in a suitable hospital waste container.
- g) The hands shall be cleaned again at the end of the use and handling of the protective coverall.

*NOTE: The protective coverall is only guaranteed protection if it respects the indicated maintenance and cleaning instructions.*

*At the end of the protective coverall's life cycle, it must be washed under the conditions defined in the section "conservation and cleaning" and disposed of as non-hazardous textile waste. If you choose not to wash the protective coverall, you must discard it as biological waste.*

### CARE AND MAINTENANCE INSTRUCTIONS

Products awaiting use must be stored in a dry, enclosed place and protected from potential contamination.

In case of defect or damage do not use the product, replace with another.

### PRODUCT LIFETIME

The protective coverall can only be used by another person after washing.

At the end of each use, the suit must be washed before being reused.

Maximum number of cycles of 25 washes in which the protective performance of the protective coverall is guaranteed.



**safe&care**

[WWW.SAFEANDCARE.PT](http://WWW.SAFEANDCARE.PT)

[geral@safeandcare.pt](mailto:geral@safeandcare.pt)

Rua do Parque Industrial n°366

4830-103 Campos PVL

PORTUGAL

T: (351) 253 639 280

FAX: (351) 253 632 600